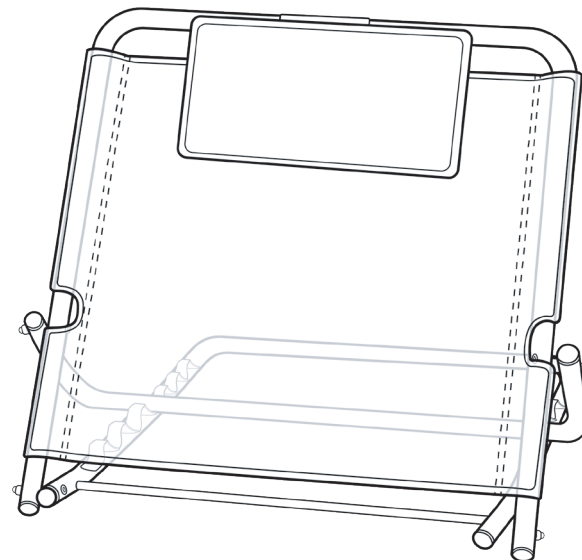
01-7000

**ATRAMA NUGARAI
SELJATUGI
BALSTS MUGURAI
BACK SUPPORT
ОПОРА ДЛЯ СПИНЫ
SPATAR
HÁTTÁMASZ
ZÁDOVÁ PODPĚRA**

Naudojimo ir priežiūros instrukcija
Lietošanas un kopšanas instrukcija
Kasutusjuhend
User manual
Руководство по эксплуатации
Manual de utilizare
Használati útmutató
Uživatelská příručka

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Observe instructions for use! Please read these instructions carefully before use and keep it for later reference!

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EN Thank you for the confidence shown in Kid-Man. Every product is factory checked and left us in faultless condition, We wish you much success and we hope Kid-Man product will make a positive impact to your daily living.

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EN Support frame is intended to be used in beds as economical solution to change the angle of backrest. Backrest is made of metal and is strong, stiff, and stable. The tilt angle is controlled mechanically, there are 6 fixed positions and the angle of backrest could be set in the range of 25-80 degrees depending on needs and comfort of user. The cushion for headrest. is removable.

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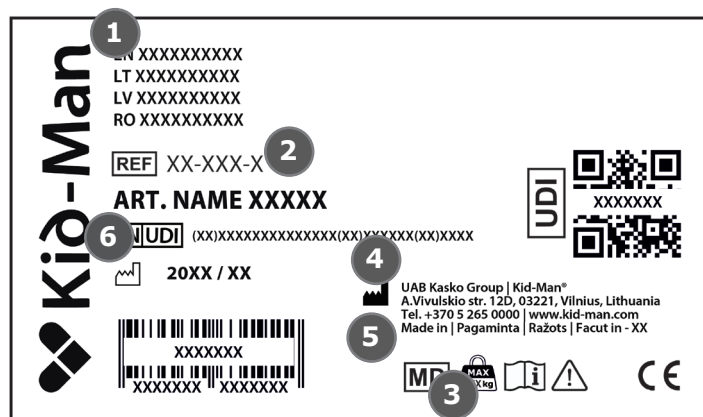
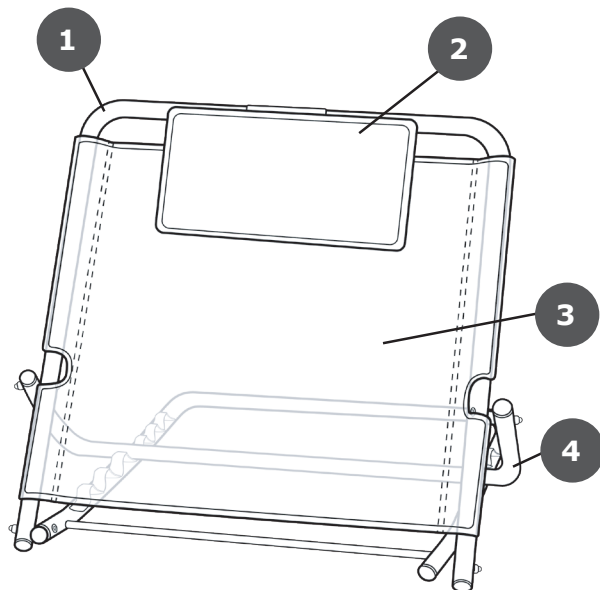
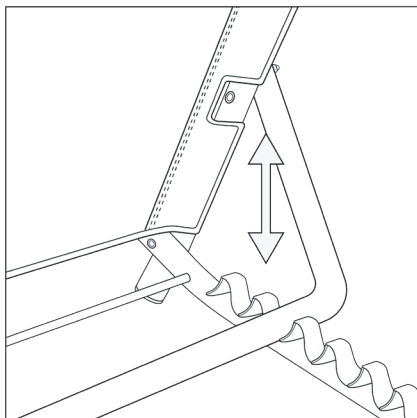
EE Toe kaldenurka muudetakse muutes toe enda kaldenurka.

EN You can adjust of tilt angle, by changing the back support angle like shown in Pic No 1.

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EN Should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent

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EN Only original spare parts can be used for replacement. Please provide model and serial number to dealer or contact manufacturer directly. Manufacturer and dealer is not responsible for any damage occurred to product or user if non original spare parts are used and (or) repair was done in unauthorized facility.

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EN We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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The quality management system of Kasko Group,
UAB meets the standards ISO 9001 and ISO 14001