**VONIOS LENTA
VANNAS DĖLA
VANNIPLAAT
BATH BOARD
ДОСКА ДЛЯ ВАННЫ
FÜRDŐKÁD PAD
SCAUN PENTRU VANA
SEDAČKA NA VANU**

02-6045C

Naudojimo ir priežiūros instrukcija
Lietošanas un kopšanas instrukcija
Kasutusjuhend
User manual
Руководство по эксплуатации
Manual de utilizare
Használati útmutató
návod k použití

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EN

PREFACE


Thank you for the confidence shown in Kid-Man. Every product is factory checked and left in faultless condition. We wish you much success and we hope Kid-Man product will make a positive impact to your daily living.

INDICATIONS


Whole structure of bath board is made of plastic thus once getting wet it could be a little slippery. Bath board is put on the sides of bath tub and is locked with feet so it could stand firmly. Before use, please check whether board is locked properly. Note, board is designed for sitting. It is not allowed to stand on it. To ensure long life time, it is recommended to check board periodically.

PARTS OF THE PRODUCT


- 1. 1 x Bath board
- 2. 4 x Feet
- 3. 4 x Feet screws
- 4. 1 x Handle
- 5. 2 x Handle screws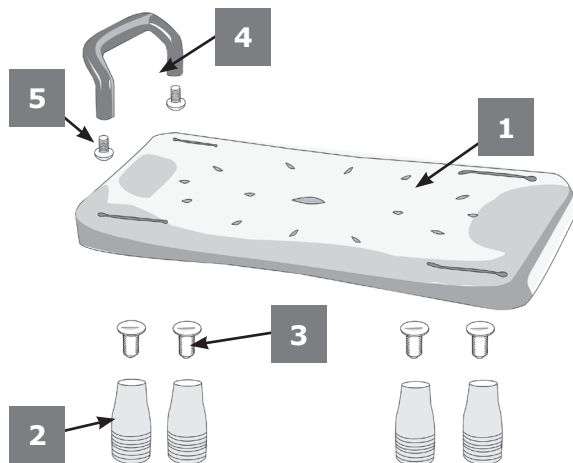

SET UP

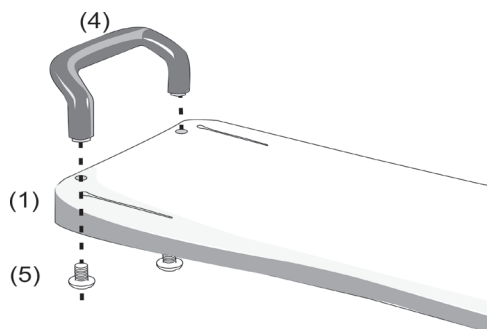
Pic. No 1

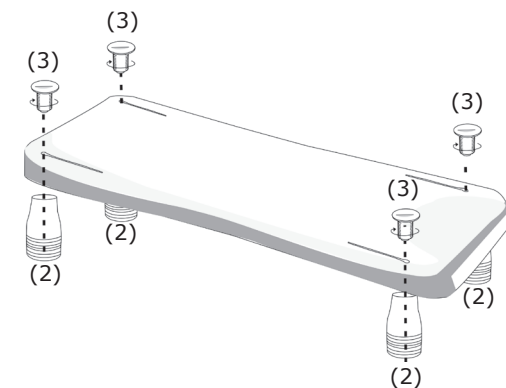Pic. No 2

Firstly, mount handle (4) using handle screws how it is shown in figure Pic. No 1. Bath board is put on the sides of bathtub. Before use, please make sure bathtub sides are flat so it could hold the construction. The minimal width of mathtub side is 38 mm (X) Pic. No. 2. Once board is applied on bathtub, please fx the feet (2) using feet screws (3). Bathboard should be orientated handle next to wall.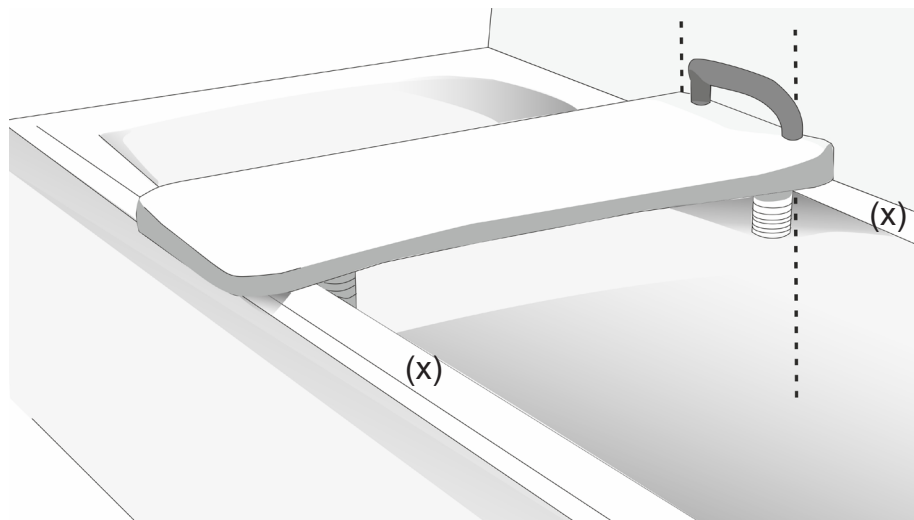


SPARE PARTS

Only original spare parts can be used for replacement. Please provide model and serial number to dealer or contact manufacturer directly. Manufacturer and dealer is not responsible for any damage occurred to product or user if non original spare parts are used and (or) repair was done in unauthorized facility.

SPECIFICATIONS

Model	02-6045C
Maximum safe load	150 kg
Lenght (A)	690 mm
Width (B)	310 mm
Min distance between feet (C)	370 mm
Max distance between feet (D)	600 mm
Weight	2,3 kg

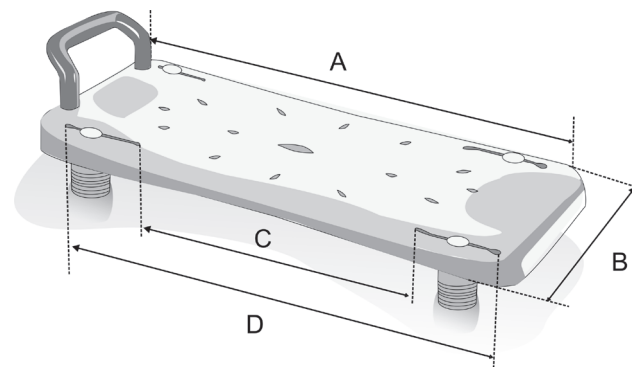Bath board 02-6045C is designed for users up to 150 kg body weight.

It is forbidden to overload bath board!

CARE AND MAINTENANCE

Clean regularly, use a general purpose, neutral detergent and warm water. Please check the suitability of the product before use. Do not use abrasive cleaners as this will permanently damage the surface and invalidate any warranty. Dry the seat thoroughly before use.

WARRANTY

We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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